

## SECTION 11

## PROCTOLOGY

### Colorectal



## IMPORTANT

1

Whenever using the Probe, screw off the Probe Sheath after treatment, wash the outside of the Sheath well in warm water with a few drops of dish washing liquid added. Then dry with a clean cloth and sterilise with colloidal silver or with pure alcohol before putting away.



Treatment times given in this user guide were designed for the home user with the Delta or Palm 980 nm laser to give a conservative starting point. Treatments with the professional 905 nm laser are more intense, and therefore these treatment times may be shortened, and also the space between treatments extended further, as determined by feeling and experience.




The Probe should be lubricated with K-Y Jelly before inserting. Do NOT use a white or an opaque cream as this will block the radiances.

## SECTION 11

## PROCTOLOGY 1

# 31 mins

Treatment time.

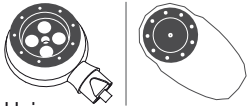

Using:  
Delta Medical Terminal or  
coMra Palm.



Using:  
Delta Probe Terminal  
with Normal Sheath.

# 1 x daily

for 15 x days


Allow 3 weeks rest and  
repeat course if needed.

### • HEMORRHOIDS

TREATMENT POINT		⌚ MINS	⚡ HZ
1	Epigastrium	2 per point	5
2	Right hypochondrium		
3	Left hypochondrium		
4	Left iliac		
5	3 points around anus		
6	Perineum		
7	Femoral arteries		
8	A If hemorrhoids are on the outside of the sphincter, place tip of the Probe against hemorrhoids	5	1000
	B If hemorrhoids are on the inside of the sphincter, insert Probe just through the sphincter		



Treatments with the professional 905 nm laser are more intense, and therefore these treatment times may be shortened, and also the space between treatments extended further, as determined by feeling and experience.

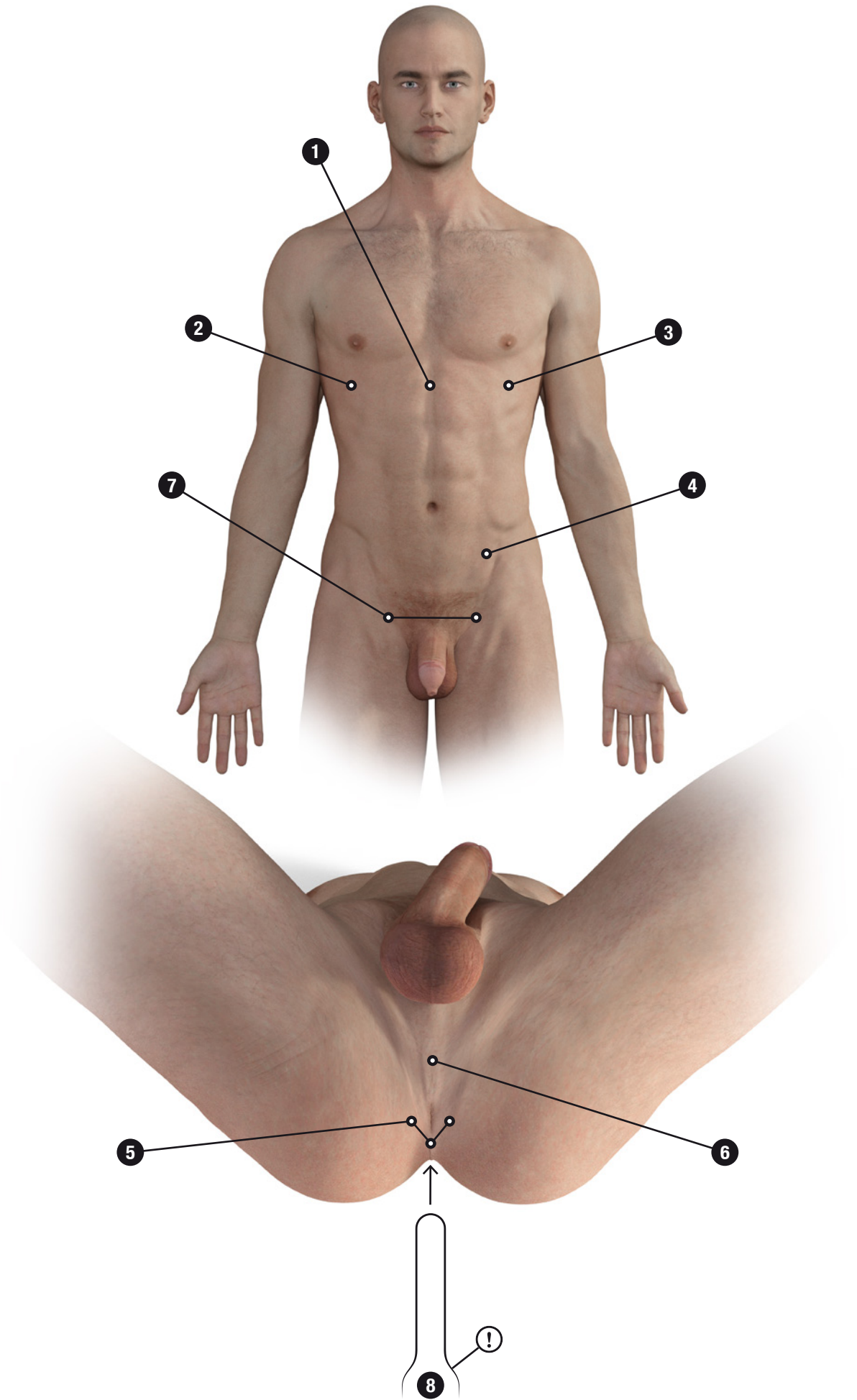


# SECTION 11

# PROCTOLOGY 2

## 11 mins

Treatment time.

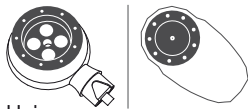

Using:  
Delta Medical Terminal or  
coMra Palm.



Using:  
Delta Probe Terminal  
with Normal Sheath.

## 1 x daily for 15 x days

Allow 3 weeks rest and  
repeat course if needed.

• **ANAL PRURITUS** – anal itching.

TREATMENT POINT	⌚ MINS	⚡ HZ
1 Scan an area of 5 cm around the anus	5	50
2 Scan both sides of vertebral column, sacral region	2 per side	1000
3 Place Probe against the anus	2	



Treatments with the professional 905 nm laser are more intense, and therefore these treatment times may be shortened, and also the space between treatments extended further, as determined by feeling and experience.




# SECTION 11

# PROCTOLOGY 3

## 12 mins

Treatment time.

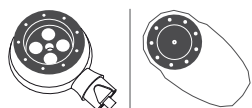

Using:  
Delta Medical Terminal or  
coMra Palm.



Using:  
Delta Probe Terminal  
with Normal Sheath.

## 1 x daily

### for 8 x days

Allow 3 weeks rest and  
repeat course if needed.

### • ANAL FISSURES

TREATMENT POINT	⌚ MINS	⚡ HZ
1 Place Probe against the anus	2	Day 1-4 @ 1000
		Day 5-8 @ Variable (V)
2 Scan an area of 5 cm around the anus with Medical Terminal	5 per frequency	5 then 50



Treatments with the professional 905 nm laser are more intense, and therefore these treatment times may be shortened, and also the space between treatments extended further, as determined by feeling and experience.

